

Control Panel at the Lift and Transport Trucks



Drive, lift, lower, load and unload. All tasks can be controlled through one centralized control panel.

The control panel can be turned on and off through the main switch (1).

The drive roller can be moved forwards (2) and backwards (3) via the arrow buttons. Through the up (4) and down (5) buttons the truck can be lifted and lowered.

The complete truck can be moved forwards and backwards through the arrow buttons (6) at the traction drive panel. The speed can be adjusted with the hand wheel (7) and can be controlled via the handle (9) at the right side of the control unit.

The traction drive is equipped with a soft start function as well as a safety lock with EMERGENCY STOP function (8) to prevent injuries of the user when driving backwards.

KUGEL medical GmbH & Co. KG
Hermann-Köhl-Str. 2a
DE - 93049 Regensburg/Germany

Phone +49/941/20 86 48-0
Fax +49/941/20 86 48-29
www.KUGEL-medical.de

