TOP NEWS

UPDATE

01

2018

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**FORMAMIX 2.0 – NOW WITH PERMANENT DISPLAY OF
FORMALIN CONCENTRATION AND ECO-FRIENDLY
ACTIVATED CARBON FILTER UNIT!**

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The International Agency for Research on Cancer (IARC) has classified formaldehyde as a class 1 carcinogen and germ cell mutagenic in 2015. For decades, formaldehyde or rather formalin has been used for tissue sample fixation in pathology. Especially during filling and opening formalin filled sample vessels, pathologists and lab staff is exposed to its harmful vapors. With the Formamix 2.0 for contact-free formalin dilution we developed its predecessor further and created a solution that manages to reduce formalin contamination in pathological laboratories to an absolute minimum.

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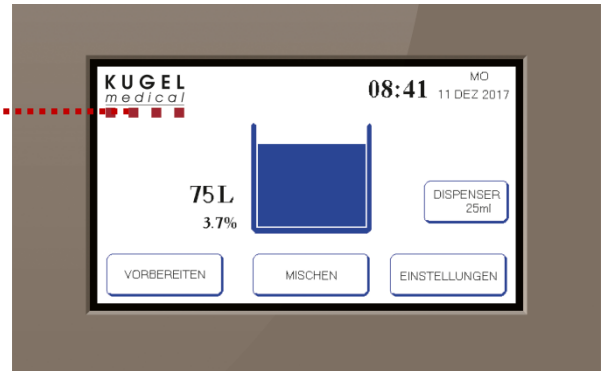
NEW FEATURES AT A GLANCE:

- Permanent display of formalin concentration in percent
- Integrated eco-friendly activated carbon filter unit
- No connection to site-mounted ventilation systems required
- Modern touch display
- Peristatic pump and foot pedal for precise dispensing of formalin to the milliliter (optional)
- Optical and acoustic warning when low level

NOW WITH PERMANENT DISPLAY OF FORMALIN CONCENTRATION AND ECO-FRIENDLY ACTIVATED CARBON FILTER UNIT!

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2. Modern Touch Display:

The new touch display makes using the Formamix 2.0 even easier. The remaining liquid quantity and formalin concentration are permanently displayed on the touch display. Just enter the desired formalin concentration and quantity. The micro processor enables you to save and recall your settings for later use. Additionally, for automatic sample vessel filling, you can set and dispense through the foot pedal the necessary formalin quantity exactly to the milliliter - without dripping or leaking!

1. Eco-friendly activated carbon filter cartridge:

The new Formamix 2.0 comes with an eco-friendly activated carbon filter cartridge. All harmful vapors are vacuumed and filtered through the filter cartridge. Optionally, you can connect the device to a site-mounted ventilation system.

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TECHNICAL DETAILS

Formamix with eco-friendly activated carbon filter:

Operation: control panel with touch display
Mains supply: 230 Volt 10 Amp / 50-60 Hz
Mains connection: shockproof plug
Power consumption: 250 W
Water connection: DN 15
Usable liquids: formalin, buffer liquid, distiller water or drinking water
Water overflow detecting sensor
Audible warning signals
Minimum liquid volume for standard solution: 10 liters
Minimum liquid volume for customized solution: 1 liter/liquid
Minimum residue liquid level required in the tank: 5 liters
Tank volume: 100 liters
Security tank volume: 67 liters
Dimensions W/L/H: 60 x 750 x 1320 mm
Dead weight: 100 kg
Ambient temperature: 5°C to 45°C
Type: AFMDS-100
Article No.: 099.900.196

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Please refer to our brochure and leaflet for more information about the reclassification of formaldehyde and our laboratory equipment with which you are able to comply with the maximum concentration values at the workplace for work involving formaldehyde and formalin!

KUGEL medical GmbH & Co. KG
Hermann-Köhl-Straße 2 A
DE – 93049 Regensburg
GERMANY

Phone +49 (0) 9 41/20 86 48-0
Fax +49 (0) 9 41/20 86 48-29
E-Mail info@KUGEL-medical.de
www.KUGEL-medical.de

KUGEL
medical
■■■■